



HomeAgain™
A lost pet's best chance

UNIVERSAL WORLDS CAN™ READER *PLUS*

Reads 134.2 kHz, 128 kHz, and 125 kHz Microchips
ISO Compatible

QUICK START GUIDE

Go to <http://microchip.homeagain.com/> for
complete User Manual.

DESCRIPTION

UWSR+ is a **universal portable reader** for RFID microchips with the purpose of identifying animals.

The device can read a **wide range of technologies**: FDX-B ISO, HDX ISO, FDX-A (FECAVA), Trovan and Avid encrypted. UWSR+ can also read and display the **Microchip Temperature** when reading microchips with temperature sensing¹ implants from Allflex or Destron Fearing.

In addition to its microchip reading functions, the UWSR+ scanner can store up to **3000 IDs** in its internal memory, each associated with a time/date stamp, which can be transmitted to a personal computer via a USB cable or wirelessly using Bluetooth®.



The UWSR+ is supplied with the Microchip Management Software which enables user to download the reader's memory, change the scanner settings or update the UWSR+ firmware. It is available for download at <http://microchip.homeagain.com/> with the complete User Manual.

¹ Microchip Temperature range is from 33°C (91.4°F) to 43°C (109.4°F). If the temperature is below or above this range, it will display "TEMP BELOW RANGE" OR "TEMP ABOVE RANGE".


GETTING STARTED

UWSR+ comes supplied with six AAA **rechargeable NiMH batteries** which must be **fully charged prior to use**, as diagram below (step 1 through 3). If preferred, six standard AAA alkaline² batteries can also be used and should not be charged (follow only steps 1 and 2).



Step 1 – Remove retention screw located on center of back of UWSR+.

Reader top

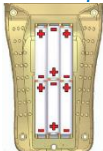

Step 2 – Place batteries according to polarity indicators as shown on diagram. Replace lid and then screw back on.

Reader bottom



Step 3 – Remove USB plug and attach cable to UWSR+.

Step 4 – Plug the adaptor into power outlet. The battery level icon will flash while charging and stop flashing when fully charged. A full charge may take approximately 3 hours.



DO NOT MIX RECHARGEABLE AND STANDARD BATTERIES
DO NOT ATTEMPT TO CHARGE STANDARD ALKALINE BATTERIES.
CHARGE THE BATTERIES INDOORS ONLY AND BETWEEN 0°C AND 40°C.

To extend battery life, charge unit at least every three months, if reader is unused.

² We recommend Duracell® or Energizer® alkaline batteries

READING MICROCHIPS

When scanning for microchips **PRESS AND HOLD BUTTON DOWN** during procedure. The reader will display "SEARCHING".

Test reader by placing test chip provided on key chain. If microchip number is not detected, consult USER MANUAL. Do not attach key chain to reader.


If a microchip is detected, the number will appear on the display. If a microchip is not detected it will display "NO ID FOUND".

The following scanning patterns are recommended to locate microchips:


For Pets

Keep scanner touching animal and scan slowly following patterns below:

Pattern #1

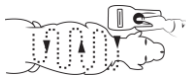

Pattern #2

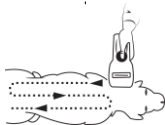

Note 1 – Microchip placement may vary per animal.

MICROCHIP MANAGEMENT SOFTWARE

UWSR+ scanner comes with Microchip Management Software only for Windows® operating system. Please refer to UWSR+ User Manual to install the tool on your PC.

Microchip Management Software offers the capabilities to:

- Change reader settings.
- Download reader memory and export animal ID's to Excel.
- Use reader as a virtual keyboard.
- Update reader firmware.

The screenshot shows the Microchip Management Software interface. At the top, there is a title bar with the text "Microchip Management Software" and a menu bar with "Connect", "Tools", and "Help". The main display area shows a large blue ID number "985141000709686" and a temperature reading "38.50°C / 101.40 °F". A "Copy" button is located to the right of the temperature. Below this, there is a table with columns for "Index", "ID code", "Date", "Time", "T(°F)", and "T(°C)". The table contains six rows of data. To the left of the table, there are three buttons: "Read memory", "Clear memory", and "Reader settings". Below these, there is a button labeled "Export to Excel". The status bar at the bottom indicates "Status: Connected COMB".

Callouts point to various features:

- Display scanned ID
- Display of microchip temperature™
- Download reader memory
- Configure your reader
- Export memory to Excel
- Choose language
- Copy ID
- Reader memory

Index	ID code	Date	Time	T(°F)	T(°C)
1	985000140023984	03/12/2015	10:24:41		
2	TR 00-0724-CEE1	03/12/2015	10:24:53		
3	985000140024612	03/12/2015	10:25:00	101.40	38.50
4	AVID*068*905*101	03/12/2015	10:25:05		
5	4C3958191B	03/12/2015	10:25:08		
6	985141000709686	03/12/2015	10:25:12	101.40	38.50

READER SETTINGS IN MICROCHIP MANAGEMENT SOFTWARE

UWSR+ is delivered with Memory and Bluetooth® disabled.

These options can be enabled through the Microchip Management Software, as in diagram below:

Reader Settings

Basic Settings

Unit ID

Silent Mode

Bluetooth ON/OFF

Temperature

Memory

Advanced Settings

Sleep Timer (s)

Bluetooth Connection Mode

Bluetooth Remote Address

Bluetooth Pin Code

Bluetooth Authentication

Transmit Time Stamp

Transmit Temperature

TROUBLESHOOTING

- **Microchip not detected:** Confirm that it is being operated distant from metal objects and computer equipment, including metal objects, such as collar with metal objects and stainless steel tables.
- **Memory cannot be downloaded:** Make sure memory is enabled in reader settings (refer to User Manual for more information).
- **Bluetooth not visible:** Check if Bluetooth icon appears on the display. If not, enable Bluetooth by using Microchip Management Software.
- **Bluetooth visible but not connectable:** Make sure your reader is configured to be visible and connectable in Microchip Management Software (refer to User Manual for more information).
- **Incorrect date/time:** Connect reader to Microchip Management Software. This will synchronize the date and time with the computer.



Note 2 – For other troubleshooting questions, consult user manual.

WARRANTY

The Universal WorldScan™ Reader Plus is warranted against defects in materials and workmanship, under normal use and service for two (2) years from the day of shipment.

This warranty will not apply if adjustment, repair or parts replacement is required because of accident, neglect, damage during transportation or causes other than ordinary use.

Manufacturer's sole responsibility under this warranty shall be at its option, to either repair or replace any product which fails during the warranty period. In no event shall Manufacturer be liable for any indirect or consequential damages or loss of profit.

LEGAL INFORMATION

Trademarks

Bluetooth® is a registered trademark of Bluetooth SIG, Inc.

Windows is a trademark or registered trademark of Microsoft Corporation in the United States and/or other countries.

All other trademarks are trademarks of their respective owners.

Apple - Legal Notice

iPhone, iPad are a trademark of Apple Inc., registered in the U.S. and other countries.

"Made for iPhone," and "Made for iPad" mean that an electronic accessory has been designed to connect specifically to iPhone, or iPad, respectively, and has been certified by the developer to meet Apple performance standards.

Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards.

Please note that the use of this accessory with iPhone or iPad may affect wireless performance.

Snapshots are according to the latest version at the moment this document was printed.

Changes may occur without notice.



HomeAgain™
A lost pet's best chance

Copyright © 2017 Intervet Inc., a subsidiary of Merck & Co., Inc. All rights reserved.

1-888-HOMEAGAIN (1-888-466-3242)

UWSR+ is a product of:

DESTRON FEARING

www.destronfearing.com



Destron Fearing™

Made for



iPhone



iPad

v.20

US/HOA/0216/0008